

SMARTLITE PRO ENDOACTIVATOR



SmartLite Pro (SLP) System



SMALL (15/02) MEDIUM (25/04)



CLINICAL APPLICATIONS

- Debridement / Smear Layer / Biofilm
- Adapt and Remove Calcium Hydroxide
- Adapt Mineral Trioxide Aggregate
- Remove Residual Obturation Remnants

SAFE • EFFECTIVE • SIMPLE

ACTIVATE YOUR ENDODONTICS

www.endoruddle.com

PRODUCT FEATURES

SLP EndoActivator Tips

- Easy snap-on/snap-off design
- Strong, flexible, medical-grade polymer composition
- 22 & 28mm lengths with conveniently placed depth gauge rings
- Parallelogram-shaped cross-section & elliptical motion for improved fluid exchange

SLP EndoActivator Handpiece

- Contra-angled design for easy access to posterior teeth
- 2-speed sonic motor for disinfection and adjunctive procedures
- Ergonomic, cordless and rechargeable



CLINICAL PROTOCOL*

- Prepare canal to an appropriately-tapered shape
- Fill pulp chamber with 17% EDTA, QMix or other final rinse solution
- Select the largest Activator tip that manually fits loosely within 2 mm of working length
- Place the barrier sleeve over the handpiece and attachment
- Snap on the Activator tip over the barrier-protected attachment
- Insert the attached Activator tip into the prepared root canal
- Depress the ON/OFF switch once for 3D disinfection (18,000cpm)
NOTE: Depress the ON/OFF switch twice for adjunctive procedures (3,000cpm)
- Initiate the disinfection cycle using a crown-down method
- Continue agitating 17% EDTA for **60 SECONDS** utilizing a gentle pumping motion
- Irrigate with a 6% solution of NaOCl, agitate for an additional **30 SECONDS**, then use intracanal suction to eliminate loose debris

*Evidence-based sequence for 3D disinfection. For other applications, see Directions For Use. For scientific evidence supporting the use of the EndoActivator, visit endoruddle.com



See Directions For Use

ADVANCED  ENDODONTICS®

Clifford J. Ruddle DDS FACD FICD
(800) 753-3636 • (805) 964-8838 • www.endoruddle.com